

**DONJOY®**

DonJoy  
Spinostrap

**BACK SUPPORT, IN HAND**



**enovis™**

**DONJOY®**

## DonJoy Spinostrap

Back support, in hand.

DonJoy Spinostrap puts prescribers and patients in control of back support. Its adjustable shoulder straps, bendable exoskeletal dorsal frame, and customizable lumbar belt enable comfortable individualized support and relief of the thoracic spine. Thanks to this semi-rigid back brace's range of intuitive features, back support and kyphotic postural correction are well within reach.

### **INTENDED USE/INDICATIONS**

DonJoy Spinostrap is designed to provide support and stabilization and partial immobilization (restriction of movement) of the spine.


- Support of stable spinal fractures
- Relief of pain and facilitation of correction of postural dysfunction from dorsal hyper-kyphosis
- Postural dysfunction associated with conditions such as Parkinsons disease
- Straightening and support of the spine in the case of muscular insufficiency/weakness
- Pain relief and support of spinal osteoarthritis



## FEATURES AND BENEFITS



**Semi-rigid exoskeletal dorsal frame:** to provide support and aid relief of the thoracic spine, aluminum core can be molded to fit the individual curvature of the back

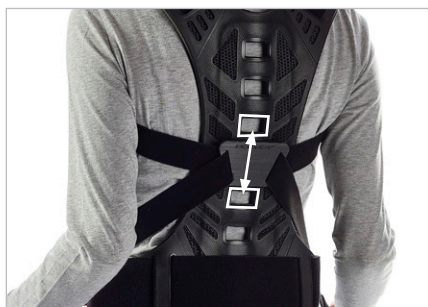


**Repositionable strap guide:** can be placed at different heights on the dorsal frame to help control the corrective action on kyphotic posture




**Adjustable lumbar belt:** uses accessible strap tabs and finger pockets to provide adjustable support for the spine while leaving the abdomen free. Can be cut to required length




**Adjustable shoulder straps:** use fixable hook-and-loop strap tabs to provide adjustable tension, aiding correction of kyphotic posture by pulling the shoulders back and straightening the back

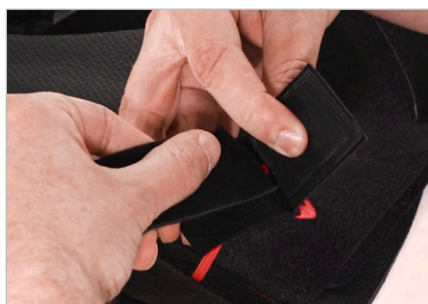

**Crocodile strap closures:** allow customization of shoulder strap and lumbar belt length for a convenient individualized fit

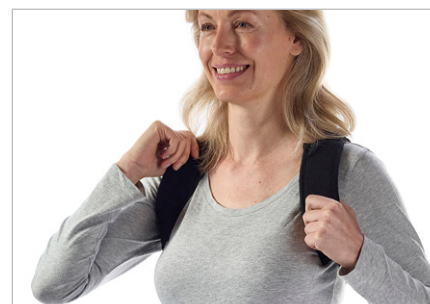

**Axillary shoulder pads:** shaped to be flat and anatomical, they can be positioned to increase the wearing comfort for the underarm area

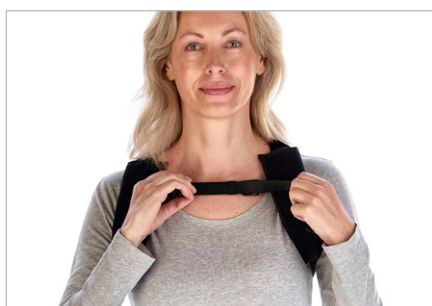

**Adjustable sternal strap:** can be attached to provide additional comfort in the axillary area



**Slim and low-profile design:** fits under clothing

**Lightweight and breathable:** helps ensure patient comfort

**Hypoallergenic and soft materials:** to help avoid skin irritation. PVC and Latex free

## ORDERING INFORMATION

PART NUMBER	SPINE LENGTH C7 – S1		SIZE
	CM	INCHES	
82-0057-1	42 – 52 cm	16½ – 20½"	S
82-0057-2	52 – 62 cm	20½ – 24½"	M
82-0057-3	62 – 72 cm	24½ – 28½"	L

# DONJOY®

## Asia Pacific

1905, Tower 2, Grand Central Plaza  
Shatin  
HONG KONG  
**T:** 852 31051415  
**E:** marketing.ap@enovis.com

## Australia – New Zealand

PO Box 6057  
Frenchs Forest DC  
NSW 2086  
AUSTRALIA  
**T:** 1300 66 77 30  
**E:** customerservice.au@enovis.com

## Benelux

Kleinhoefstraat 5, bus 39  
Geel 2440  
BELGIUM  
**T:** +32 (0)14 248090  
**E:** orders.bx@enovis.com

## Canada

6485 Kennedy Rd  
Mississauga  
Ontario, L5T2W4  
CANADA  
**T:** 1-866-866-5031  
**E:** canada.orders@enovis.com

## China

A312 SOHO ZhongShan Plaza  
1055 W. ZhongShan Rd  
Shanghai, 200051  
CHINA  
**T:** (8621) 6031 9989  
**E:** shujun.tang@enovis.com

## France

3 rue de Bethar  
Centre Européen de Frêt  
64990 Mouguerre  
FRANCE  
**T:** +33 (0)5 59 52 86 90  
**E:** sce.cial@enovis.com

## Germany

Bötzingen Straße 90  
79111 Freiburg  
GERMANY  
**T:** +49 (0) 180 1 676 333  
**E:** kundenservice@enovis.com

## Italy

Via Leonardo da Vinci 97  
Trezzano sul Naviglio  
Milan 20090  
ITALY  
**T:** +39 02 835 98001  
**E:** vendite@enovis.com

## Nordic

Murmansgatan 126  
212 25 Malmö  
SWEDEN  
**T:** +46 40 39 40 00  
**E:** info.nordic@enovis.com

## South Africa

Unit 6, 2 on London, 2 London Circle  
BrackenGate Business Park  
Brackenfell, 7560  
SOUTH AFRICA  
**T:** +27 (0) 21 276 2968  
**E:** info.southafrica@enovis.com

## Spain

Av.Cornella 144. 1-4  
08950 Esplugues de Llobregat  
Barcelona  
SPAIN  
**T:** +34 93 480 32 02  
**E:** ventas@enovis.com

## UK & Ireland

1a Guildford Business Park  
Guildford, Surrey  
GU2 8XG  
UK  
**T:** +44 (0)1483 459 659  
**E:** ukorders@enovis.com

## USA

2900 Lake Vista Drive  
Lewisville  
TX 75067  
U.S.A.  
**T:** 760 727 1280  
**E:** customercare@enovis.com

**enovis**<sup>™</sup>

*Creating Better Together™*

**enovis-medtech.eu**

1a Guildford Business Park, Guildford, Surrey, GU2 8XG, UNITED KINGDOM

Individual results may vary. Neither Enovis, DJO, LLC or any of their subsidiaries dispense medical advice. The contents of this document do not constitute medical, legal, or any other type of professional advice. Rather, please consult your healthcare professional for information on the courses of treatment, if any, which may be appropriate for you.

MKT-000-00171850-EN - Rev A

Copyright © 2023 DJO, LLC, a subsidiary of Enovis Corporation