

enovis™

DONJOY®

OA GO®

PAIN RELIEF WITH A TWIST.



DONJOY®

OA GO®

PAIN RELIEF WITH A TWIST.

We all know the importance of staying active, particularly as we grow older, but when pain stands in the way, sometimes that can be easier said than done.

OA GO® from DonJoy® helps provide relief from pain and mobility issues caused by mild to moderate knee osteoarthritis. With a simple twist of a dial, this innovative soft knee brace's three-point unloading system quickly unloads the affected compartment to help relieve pain and ease movement.^{1,2}

OA GO is a low profile, lightweight combination of sleeve and wrap that slides over the leg for easy application. Its flexible sidebar connects to a system of cables controlled by dual BOA® dials—just turn to achieve the necessary level of unloading and you're ready to go.

That's why, whether activity means rambling in the countryside, climbing the steps of a gallery, or playing in the garden with the grandkids, this brace is an invitation to go for it.

INTENDED USE/INDICATIONS

- Mild to moderate single compartment osteoarthritis (medial or lateral)
- General instability of the knee joint by age
- Varus-valgus knee conditions that may benefit from unicompartement unloading

1. Ramsey DK, Russell ME. Unloader Braces for Medial Compartment Knee Osteoarthritis: Implications on Mediating Progression. *Sports Health - A Multidisciplinary Approach*. 2009;1(5):416-426.

2. Mistry DA, Chandratreya A, Lee PYF. An Update on Unloading Knee Braces in the Treatment of Unicompartemental Knee Osteoarthritis from the Last 10 Years: A Literature Review. *Surg J (N Y)*. 2018 Jul 2;4(3):e110-e118



FLEXIBLE SHELLS

BOA® DIAL

CONDYLE PAD

FLEXIBLE HINGE

BOA DIAL

FLEXIBLE SHELLS



FEATURES AND BENEFITS

COMFORT



Anatomically contoured condyle pads: Adhere to the knee anatomy for improved pressure distribution. Two different thicknesses (10mm and 13mm) of condyle pad included to aid individual fit and comfort for varied anatomies.



Anti-slip bands: Positioned at the top of the thigh to prevent brace migration and maintain a comfortable fit.



Breathable, stretchable, antibacterial fabrics: Aid comfort, compliance, and hygiene, enabling extended periods of use. Not made with natural rubber latex. Rear Lycra section provides additional comfort and freedom over the popliteal fossa.

EASE OF USE




Hybrid sleeve/wrap design: Half wrap, half sleeve, OA GO® combines the benefits of both to make fitting quick and easy. Simply open the thigh section then pull the lower sleeve section over the calf before securing the support with the hook-and-loop fastening.



Precision fit and fast release: Twin multidirectional BOA® Li2 dials deliver a precision fit engineered to perform in the toughest conditions. Turn clockwise and counter-clockwise for independent tightening and releasing of thigh and calf areas, or pull up for quick release.



Easy-grip pull tab: Facilitates quick and easy attachment and release of the locking plate, and folds down flat to avoid catching.

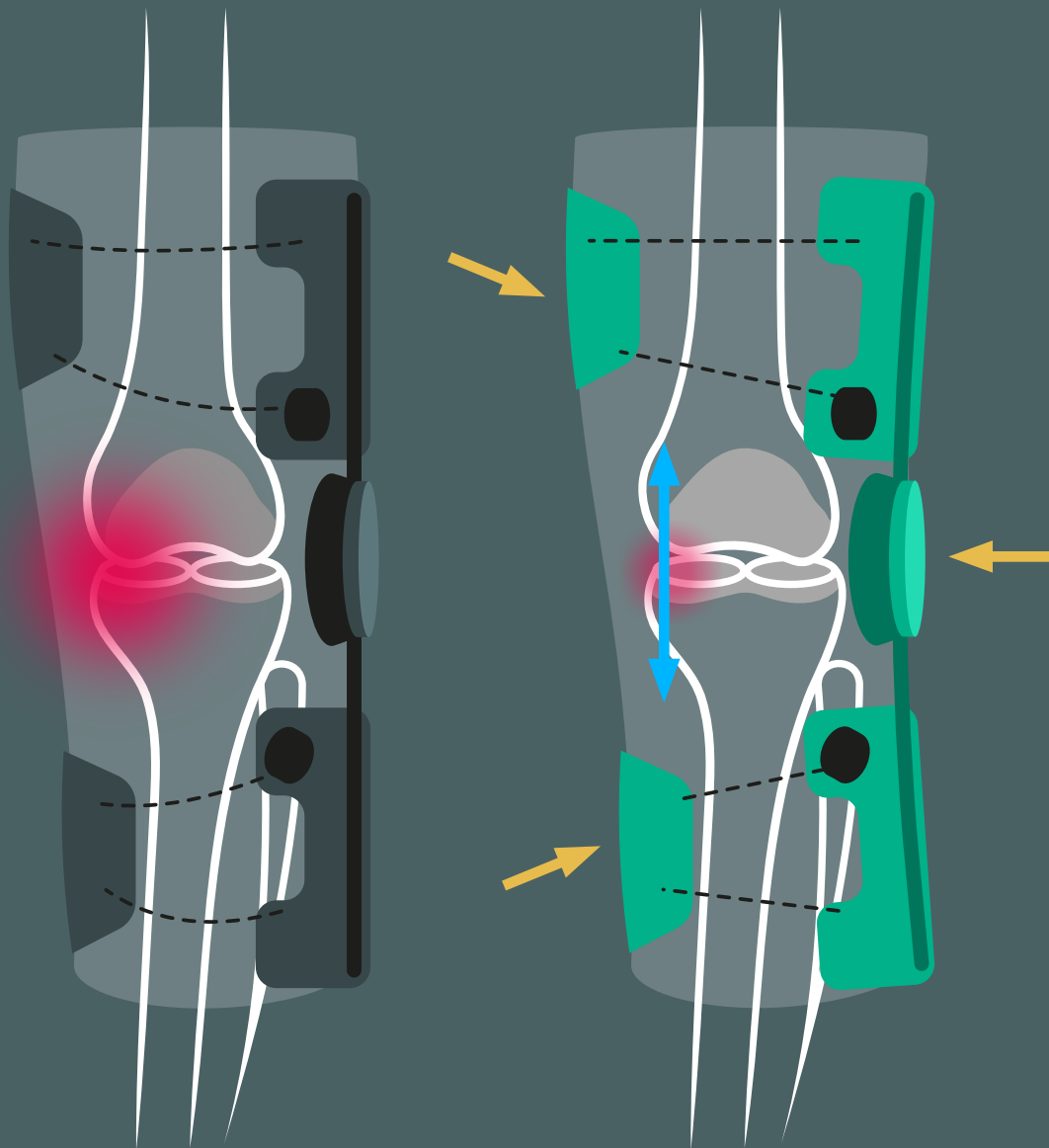


UNIQUE 3-POINT LEVERAGE SYSTEM

OA GO's patented fitting system uses adjustable BOA® dials to control the level of tension within its two cable loops. This tension reorients the flexible elements of the brace to create a three-point leverage system that offloads the affected compartment and helps provide relief.

ORDERING INFORMATION

PART NUMBER		THIGH CIRCUMFERENCE		SIZE
RIGHT MEDIAL/ LEFT LATERAL	LEFT MEDIAL/ RIGHT LATERAL			
82-7428-1	82-7429-1	33 – 39 cm	13 – 15 ½"	XS
82-7428-2	82-7429-2	39 – 47 cm	15 ½ – 18 ½"	S
82-7428-3	82-7429-3	47 – 53 cm	18 ½ – 21"	M
82-7428-4	82-7429-4	53 – 60 cm	21 – 23 ½"	L
82-7428-5	82-7429-5	60 – 67 cm	23 ½ – 26 ½"	XL
82-7428-6	82-7429-6	67 – 75 cm	26 ½ – 29 ½"	XXL

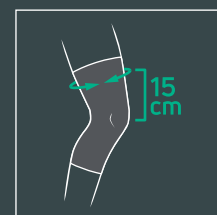

Measurement taken 15 cm above mid-patella



DONJOY®

Asia Pacific

1905, Tower 2, Grand Central Plaza
Shatin
HONG KONG
T: 852 31051415
E: marketing.ap@enovis.com

Australia – New Zealand

PO Box 6057
Frenchs Forest DC
NSW 2086
AUSTRALIA
T: 1300 66 77 30
E: customerservice.au@enovis.com

Benelux

Kleinhoefstraat 5, bus 39
Geel 2440
BELGIUM
T: +32 (0)14 248090
E: orders.bx@enovis.com

Canada

6485 Kennedy Rd
Mississauga
Ontario, L5T2W4
CANADA
T: 1-866-866-5031
E: canada.orders@enovis.com

China

A312 SOHO ZhongShan Plaza
1055 W. ZhongShan Rd
Shanghai, 200051
CHINA
T: (8621) 6031 9989
E: shujun.tang@enovis.com

France

3 rue de Bethar
Centre Européen de Frêt
64990 Mouguerre
FRANCE
T: +33 (0)5 59 52 86 90
E: sce.cial@enovis.com

Germany

Bötzingen Straße 90
79111 Freiburg
GERMANY
T: +49 (0) 180 1 676 333
E: kundenservice@enovis.com

Italy

Via Leonardo da Vinci 97
Trezzano sul Naviglio
Milan 20090
ITALY
T: +39 02 835 98001
E: vendite@enovis.com

Nordic

Murmansgatan 126
212 25 Malmö
SWEDEN
T: +46 40 39 40 00
E: info.nordic@enovis.com

South Africa

Unit 6, 2 on London, 2 London Circle
BrackenGate Business Park
Brackenfell, 7560
SOUTH AFRICA
T: +27 (0) 21 276 2968
E: info.southafrica@enovis.com

Spain

Av.Cornella 144. 1-4
08950 Esplugues de Llobregat
Barcelona
SPAIN
T: +34 93 480 32 02
E: ventas@enovis.com

UK & Ireland

1a Guildford Business Park
Guildford, Surrey
GU2 8XG
UK
T: +44 (0)1483 459 659
E: ukorders@enovis.com

USA

2900 Lake Vista Drive
Lewisville
TX 75067
U.S.A.
T: 760 727 1280
E: customercare@enovis.com

enovisTM

Creating Better Together™

enovis-medtech.eu

1a Guildford Business Park, Guildford, Surrey, GU2 8XG, UNITED KINGDOM

Individual results may vary. Neither Enovis, DJO, LLC or any of their subsidiaries dispense medical advice. The contents of this document do not constitute medical, legal, or any other type of professional advice. Rather, please consult your healthcare professional for information on the courses of treatment, if any, which may be appropriate for you. BOA® is a registered trademark of BOA® Technology Inc.